40
42
44

Strumenti per legature

Ligature instruments

40

164

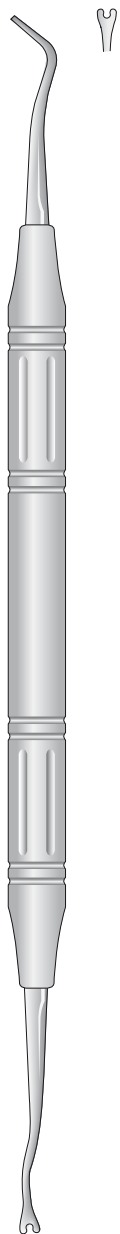
fig. 1

4000.##.01

fig. 2

4000.##.02

fig. 7

4000.##.07

fig. 11

4000.##.11

fig. 10

4000.##.10


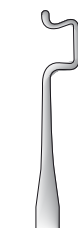
fig. 20

4000.##.20

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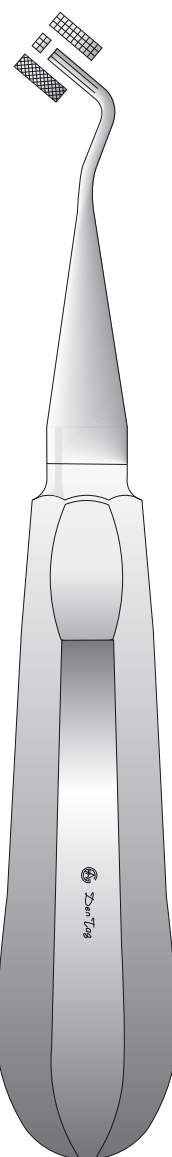Elastic instrument

Mershon

4001.Q0.01

##

A0

G0


DT06

DT08

DT10

Manici disponibili

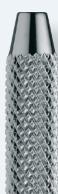Available handles

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Dental mouth mirrors allow the dentist or hygienist to see areas of the mouth that are otherwise difficult to visualize directly, such as the back of the teeth or the upper palate.

They reflect light into hard-to-see areas, improving visibility inside the mouth.

They help retract the tongue, cheeks, or lips to provide a clear view and better access to the areas to be treated.

They offer protection by gently moving soft tissues, such as the cheeks, away from the treated area, preventing injury.

In summary, **dental mirrors are simple yet indispensable tools in dentistry**, both for routine exams and more complex procedures.

The shape and position of the reflective surface affect how the mirror reflects light and provides visibility.

Specchietti e manici per specchietti Mouth mirrors and handles and mouth mirrors

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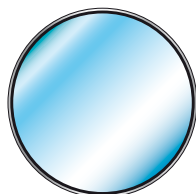
Flat mirrors

The reflective surface is placed behind the glass. It produces a slightly reduced image due to the space between the mirror surface and the glass.

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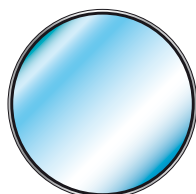
Concave mirrors

The reflective surface is concave, like a lens, magnifying the image and offering a larger, closer view.

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Mouth mirror with front
reflective surface on
both sides

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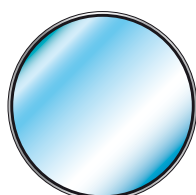
Front-surface mirrors

The reflective coating is located on the front surface of the mirror. This avoids double reflections and distortions.

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It is also important to consider that the thread type must be the same on both the mirrors and the handles.

The mirrors follow two international standards, Simple Stem (S.S.) or European threading, and Cone Socket (C.S.) or American threading.

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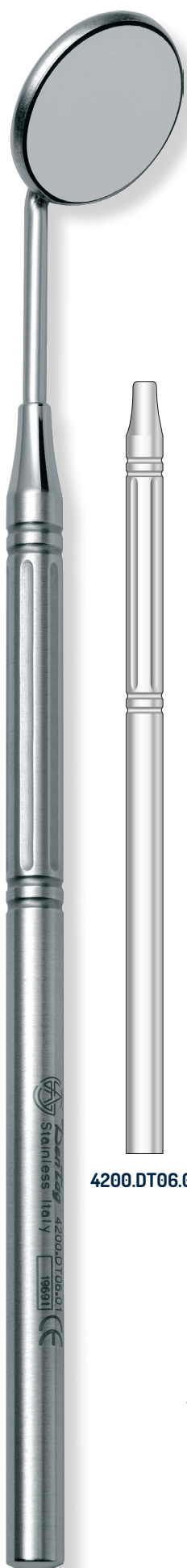
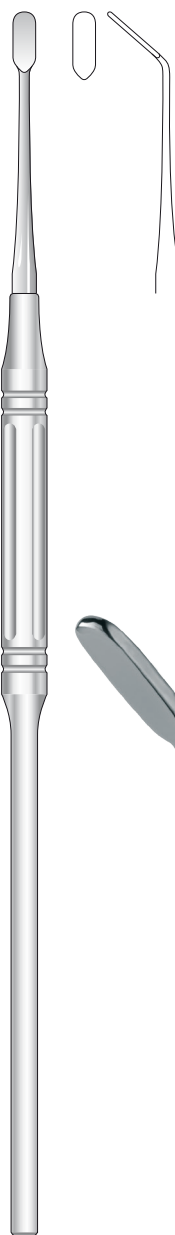
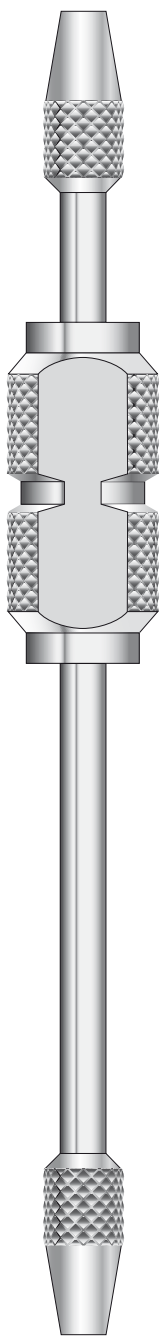
The double-bladed scalpel handles allows for parallel cuts in a single motion. Thanks to the arrangement of the blades, it is possible to perform a homogeneous connective tissue graft along the entire length. By using the scalpel blades oriented parallel to the hard palate, a precise amount of tissue can be removed in minimal time, reducing the risk of perforation and minimizing discomfort for the patient.